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**NOTE!** The complete installation of the system is described in the Technical Manual.

**CAUTION!** The printed circuit board includes electrostatic sensitive components. Avoid touching.

## Connection socket with call switch, article no. 70 0171 60...\*)

...\*) The letter at the end of the article no. represents the frame of the switch:

A = Frame dimensions (HxW): 91 x 91 mm

F = Frame dimensions (HxW): 80 x 80 mm

### Intended use

Connection socket including two identical 8-pin sockets for call devices (e.g. pear push switch) with 8-pin mini-DIN plug.

One call button complete with integrated reassurance light. The reassurance light illuminates upon activation of a call.

The connection of an additional external call device incl. location light and reassurance light is made on the back side with a special connection cable (50 0308 02).

2 switching outputs for light switching: 24 V DC, max. 40 mA per switching output.

Pressing the call button, triggering a connected pear push switch or triggering the call device connected to the external call input triggers the "Call" call type. An emergency call is triggered when presence is activated.

## Connection socket with call switch, WC article no. 70 0171 62...

As 70 0171 60..., except for the following differences:

Pressing the call button, triggering a connected pear push switch or triggering the call device connected to the external call input triggers the "WC call" call type. A WC emergency call is triggered when presence is activated.

## Connection socket with call switch, Alarm article no. 70 0171 65...

As 70 0171 60..., except for the following differences:

Pressing the blue alarm button of a pear push switch 50 067 00 triggers the "Alarm" call type. Pressing the red call button of this pear push switch triggers the "Call" call type. NOTE! The pear push switch 50 1067 00 can only be used in conjunction with connection socket 70 0171 65 A.

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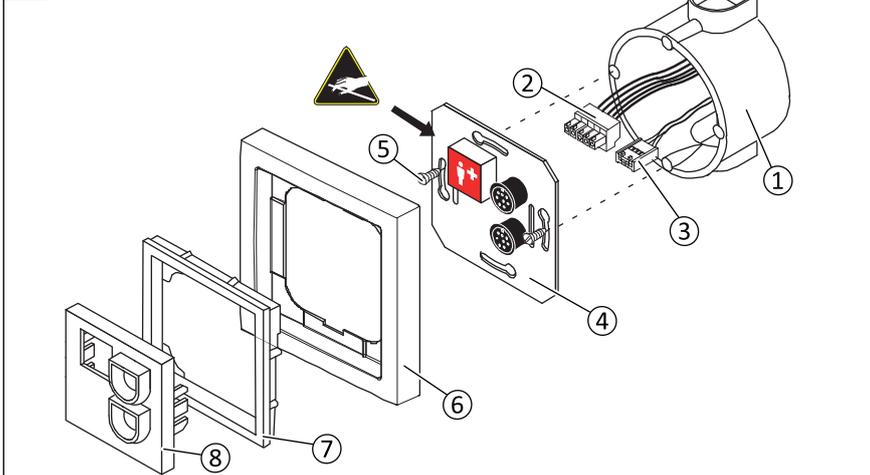
**External call device:**


ON	5	6	= No external call device connected.
	5	6	= Normally open contact.
	5	6	= Normally closed contact.

Coding switch 4 must always be set to **OFF**.

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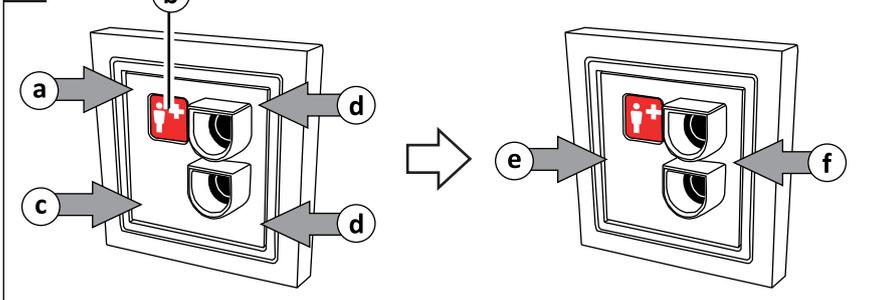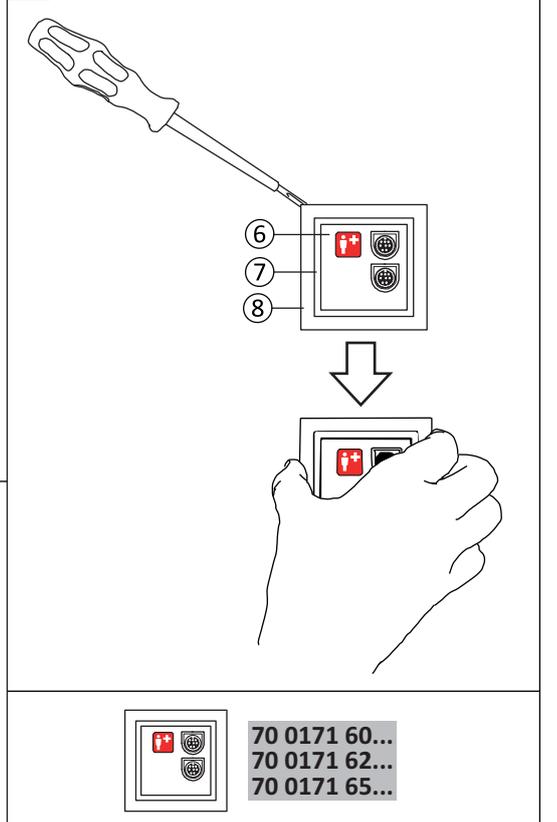
\* Not included in the scope of delivery, please order separately.

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**Mounting (→ fig. A, B, C)**

1. Plug the 5-pole connector [2] into the 5-pole socket on the rear side of the PCB [4].
2. If an external call device shall be connected, plug the connector of the connection cable [3] for the external call device into the appropriate socket on the rear side of the PCB [4].
3. Screw the PCB [4] to the back box [1] with the back box screws [5].
4. Position the central plate [8] together with the intermediate frame [7] and the frame [6] in front of the PCB [4].
5. Fix the central plate [8] with the intermediate frame [7] and frame [6] on the PCB. To prevent the red call button from wedging or breaking off, proceed as follows according to fig. C:
  - a) Press on the top left of the central plate [8], but do not yet engage it.
  - b) Carefully guide the red call button through the opening in the central plate [8].
  - c) Press the central plate [8] down on the left, but do not yet engage it.
  - d) Press on the top and bottom right of the central plate [8], but do not yet engage it.
  - e) Press the left side of the central plate [8] firmly until it audibly engages.
  - f) Press the right side of the central plate [8] firmly until it audibly engages.

**Dismantling (→ fig. D)**

1. Lever the frame [6] together with the intermediate frame [7] and the central plate [8] off the wall. To do this, proceed as follows:
  - a) Using a slotted screwdriver (slot width 8 - 10 mm), lift the four corners one after the other by approx. 4 mm.
  - b) Pull off the frame [6] together with the intermediate frame [7] and the centre plate [8] by hand.
2. Undo the screws [5] and remove the PCB [4] from the back box [1].
3. Unplug the connectors from the rear of the PCB [4].

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Quiescent current consumption	21 mA
Central plate colour	Studio white, similar to RAL 9016
Central plate material	Polycarbonate

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...\*) The letter at the end of the article number represents the frame of the switch:

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Pressing the call button, triggering a connected pear push switch or triggering the call device connected to the external call input triggers the "WC call" call type. A WC emergency call is triggered when the presence is activated.

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




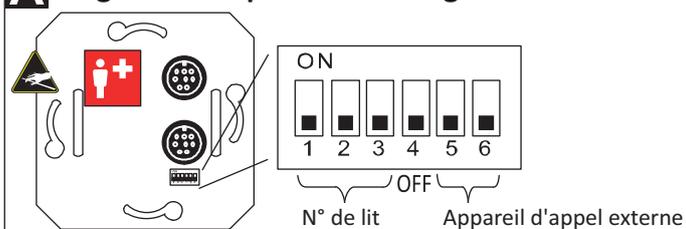
**A Setting coding switches**

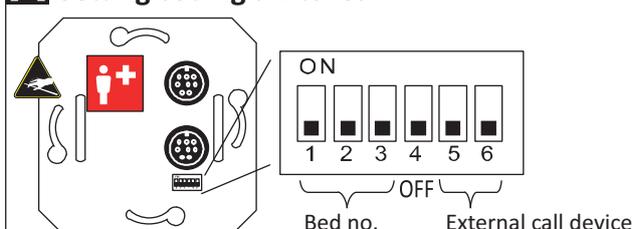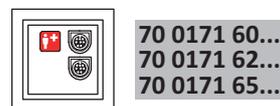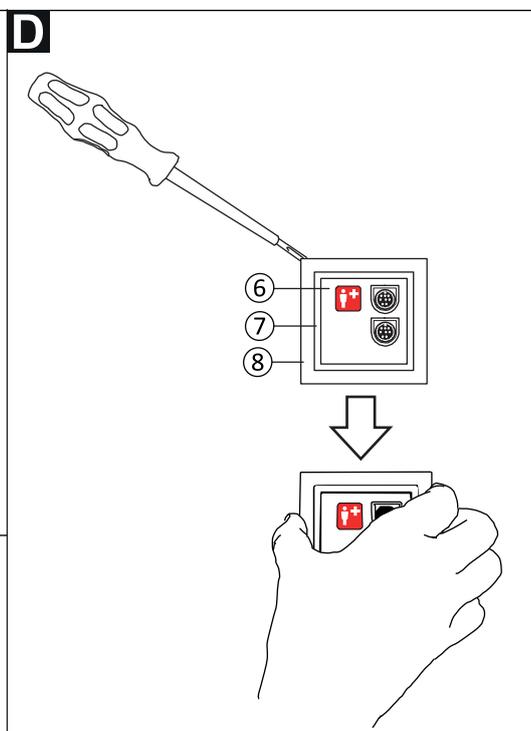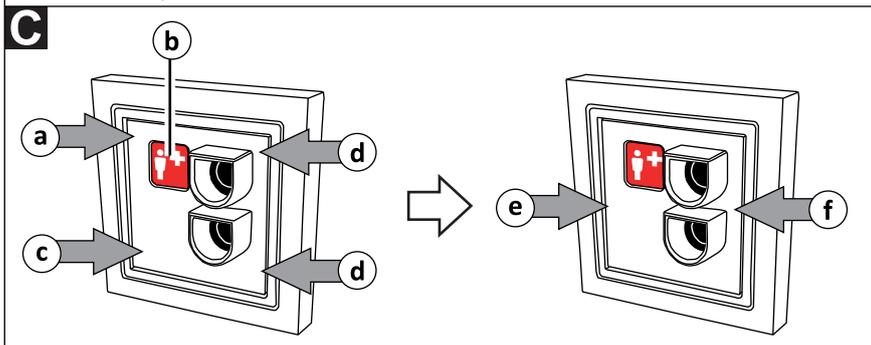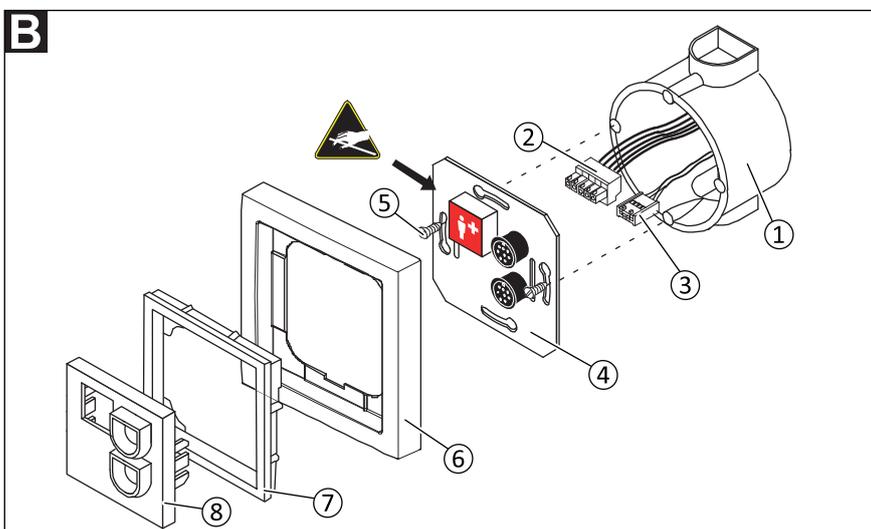- Bed no.:**
- ON    = Bed 1
  - = Bed 2
  - = Bed 3
  - = Bed 4
  - = Bed 5
  - = Bed 6
- External call device:**
- ON   = No external call device connected.
  - = Normally open contact.
  - = Normally closed contact.
- Coding switch 4 must always be set to OFF.

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Connections (→ fig. E)

As external call device a normally-closed or a normally-open contact may be used. The external call device is connected via the special connection cable, article no. 50 0308 02 (length: 50 cm). The maximum cable length for the external call device is 2.5 m.

<p><b>E</b></p>	
<p><b>Raccorder RAN et éclairage :</b></p>	<p><b>Connecting RAN and light:</b></p>
<p><b>Raccorder appareil d'appel comme contact à ouverture :</b></p>	<p><b>Connecting external call device as normally closed contact:</b></p>
<p><b>Raccorder appareil d'appel externe comme contact à fermeture:</b></p>	<p><b>Connecting external call device as normally open contact:</b></p>

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© Tunstall GmbH, Orkotten 66, 48291 Telgte, Germany, www.tunstall.de